

## Monkeypox Antigen Rapid Test

Monkeypox is an infectious disease caused by the monkeypox virus that can occur in certain animals, including humans. Symptoms begin with fever, headache, muscle pains, swollen lymph nodes, and feeling tired. This is followed by a rash that forms blisters and crusts over. The time from exposure to onset of symptoms is usually 7 to 14 days. The duration of symptoms is typically two to four weeks. Humans can be infected by an animal via a bite, or by direct contact with an infected animal's bodily fluids. The virus can also spread from human to human, by respiratory (airborne) contact or by contact with an infected person's bodily fluids.

- Get results in just 15 minutes
- High Sensitivity
- Reliable Result



## Intended Use

Monkeypox Antigen Rapid Test Cassette is a solid phase immunochromatographic assay for the rapid, qualitative and differential detection of monkeypox virus antigen in human throat swab or vesicle /acne scab focus swab specimen. This test provides only a preliminary test result. Therefore, any reactive specimen with the Monkeypox Antigen Rapid Test Cassette must be confirmed with alternative testing method(s) and clinical findings.

## Materials Supplied

For 1/2/5 Test(s)/Kit


- 1/2/5 Test cassettes
- 1/2/5 Sterile swabs
- 1/2/5 Extraction tubes with buffer and tips
- 1 Package insert

For 7/20 Test(s)/Kit

- 7/20 Test cassettes
- 7/20 Sterile swabs
- 7/20 Extraction tubes with buffer and tips
- 1/2 Workstations
- 1 Package insert



## Test Procedure

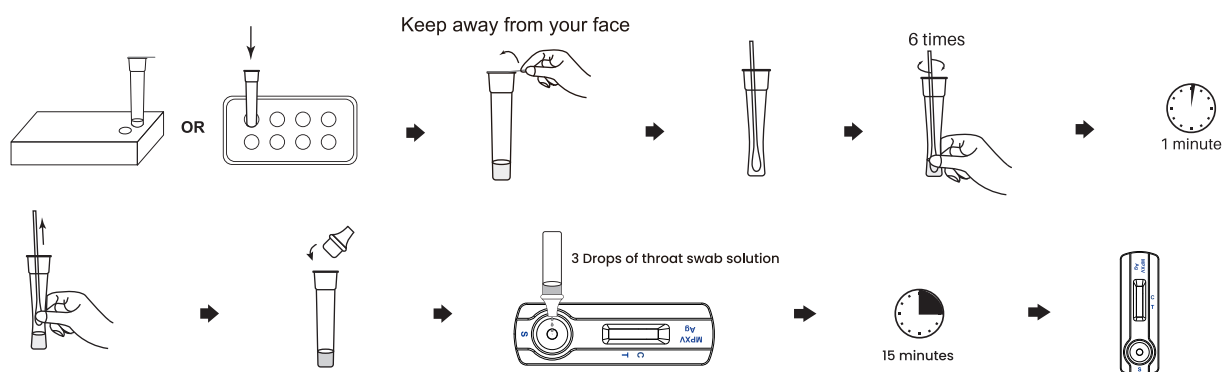

## Interpretation

**POSITIVE**  

 The presence of two lines, the control line (C) and test line (T), within the result window indicates a positive result.

**NEGATIVE**  

 The presence of only the control line (C) within the result window indicates a negative result.

**INVALID**  

 If the control line (C) is not visible within the result window after performing the test, the result is considered invalid.

## Ordering Information

Product Description	Specimen	Catalog No.	Format	Kit Size
Monkeypox Antigen Rapid Test	Throat swab or vesicle / acne scab focus swab	GCMKP-502b √	Cassette	1/2/5/7/20 Test(s)/Kit

√ CE Marked

Revision date:2022-09



Zhejiang Orient Gene Biotech Co., Ltd  
 Add: 3787#, East Yangguang Avenue, Dipu Street,  
 Anji, Huzhou, Zhejiang, China  
 Tel: +86-572-5303755/5303756  
 Fax: +86-572-5226222  
 Web: <http://www.orientgene.com>