


Digital Pregnancy Test

Key Facts


Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta shortly after the conception and secreted into the urine and serum. hCG can be detected in both urine and serum as early as 7 to 10 days after conception. hCG levels continue to rise very rapidly, frequently exceeding 100 mIU/mL by the first missed menstrual period, and peaking in the 100,000 - 200,000 mIU/mL range about 10 - 12 weeks into pregnancy.

hCG LEVEL DURING PREGNANCY

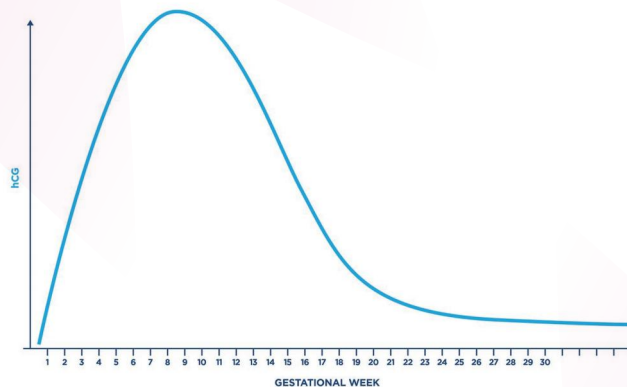

Introduction

The Digital Pregnancy Test is a rapid, qualitative detection of human chorionic gonadotropin ("hCG") in urine, which is suitable for the early diagnosis of pregnancy. The equipment is self-test IVD medical equipment which is intended by the manufacturer for use by lay person in a home environment.

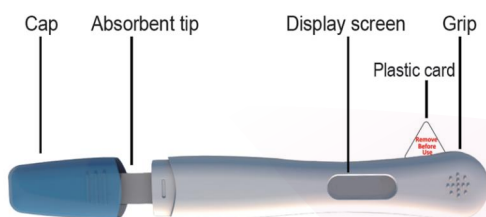

Benefits

- Accuracy greater than 99%.
- Simple test procedure for self-test.
- Visualization screen for intuitive clarity.
- Fast results within 5 minutes.

Test Procedure

1. Remove the plastic card follow the arrow direction (Refer to Fig. 1)
2. Hold the Absorbent Tip pointing downwards in your urine stream (Refer to Fig.2) Or Collect your urine into a clean and dry cup and dip the whole absorbent tip into the urine (Refer to Fig.3)
3. The Hour Glass symbol will appear on the display screen (Refer to Fig. 4)
4. In about 10 seconds, the Hour Glass symbol will begin to flash, and then remove the device from the urine stream and keep the Absorbent Tip pointing downwards or lay the Test Device flat.
5. Within 5 minutes, the Hour Glass symbol stops flashing, and then the screen will show your result.



Fig.1



Fig.2



Fig.3




Fig.4

Interpretation Of Results




POSITIVE

When the screen displays "Pregnant", it indicates that you are pregnant first 10 days. If the display screen displays "Pregnant > 10", it indicates that you are pregnant between 11 and 28 days.

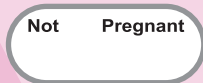

NEGATIVE

When the screen displays "Not Pregnant", it indicates that no pregnancy has been detected.

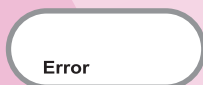

INVALID

When the screen displays "Error", it indicates that the test fails, is invalid or is used. Repeat with a new test.

Ordering Information

Product Description	Specimen	Catalog No.	Format	Cut-off Value	Kit Size
Digital Pregnancy Test	Urine	GAHCG-D103a√	Midstream	25 mIU/mL	1/2 Test(s)/Kit

√CE Marked

Zhejiang Orient Gene Biotech Co., Ltd
 Add: 3787#, East Yangguang Avenue, Dipu Street,
 Anji, Huzhou, Zhejiang, China
 Tel: +86-572-5303755/5303756
 Fax: +86-572-5226222
 Web: <http://www.orientgene.com>

Revision date: 2021-09-02